

## BEST PRACTICE SUBMISSION GUIDELINES

# MYCOPLASMA HYOPNEUMONIAE TRACHEAL SWABS

## PACKING SUPPLIES:


- 50ml conical vials
- PBS – 15ml per vial
- PCAI rod
- Scissors
- Sealable bag
- Sharpie
- Cooler
- Ice packs

## Additional Collection Supplies:

- Headlamp
- Laryngoscope
- Oral speculum
- Snare


1.

Place 15ml  
PBS in a 50ml  
conical vial

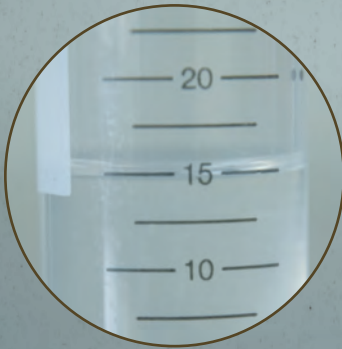

2.

Collect sample and cut  
tip as short as possible.

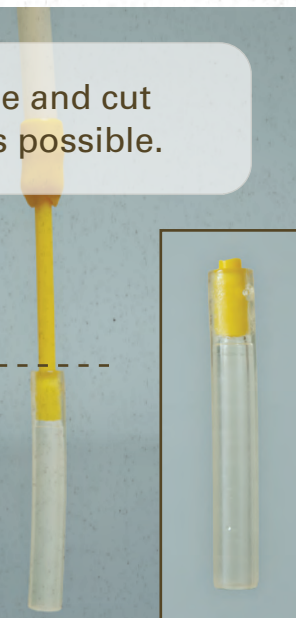

3.

Place **up to 5**  
**cut tips** into  
15ml of PBS  
in conical vial.  
Ensure vial is  
closed.



4.

Package labeled vials  
in a sealable bag.

